

SMP AT - Digital Alcohol Tester

How it works:

Switch-on time is 10 seconds, in which case the device starts counting down, from 10 seconds to 0. In this case, it adapts to the outside temperature. Once this is done, it will beep and start counting down again from 10 seconds. Then blow the probe with a lot of air, evenly, not too hard, until the device beeps. After the beep, the measured values appear on the display.

Displays "CAUTION" if the result is between 0.02% and 0.05%.

Displays "DANGER" when the result reaches or exceeds 0.05%.

Measuring range 0.00% to 0.15% or 0.0 to 1.9 g/l

Displays "OFF" 10 seconds after the measurement is completed, then turns off automatically after about 3 seconds.

Measurement countdown 10 sec

Works with 2 x AAA batteries.

Dimensions 11 cm x 7 cm x 2.5 cm.

Important: Do not smoke, eat or drink alcohol for at least 20 minutes before starting the measurement.

Technical information:

Digital alcohol probe with audible alarm

Sensing limits: 0.00-0.19% BAC / 0.00-1.9 g/l

Fast response and reusability

Auto power off function

5 replacement blower tubes

Keychain

Operation: with 2 AAA batteries (battery not included)

Reference:

BAC: Blood Alcohol Concentration

0.05% BAC: Equivalent to 0.05g alcohol per 100ml blood unit

Warning: The product is not a certified device, so the measured values are for information only !!

Importer;

S.M.Power Kft

1310 Szigetszentmiklós, Csepeli út 15

www.smpower.eu